

How to remove FreedomChair batteries




- 1** ** Before removing any connections, take a photo for reference. Unscrew the locking ring on the bottom of the battery until it's completely loose.




- 2** Carefully pull the plug straight out until it fully detaches from the battery.




- 3** Place your hand under the battery and try to find the clamp (see picture 5). Squeeze and gently pull the battery out.

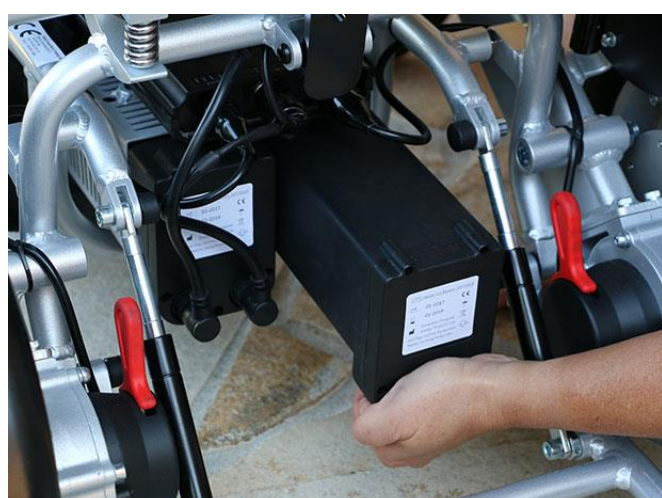

- 4** Continue to pull the battery outward until it fully detaches from the holder. Store the battery cautiously.

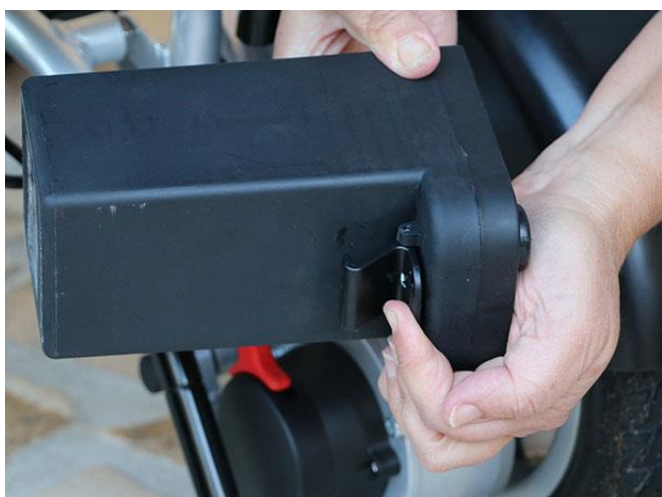

- 5** On the underside of the battery there is a clamp that holds it in place in its holder. To release the battery, the clamp must be pressed.

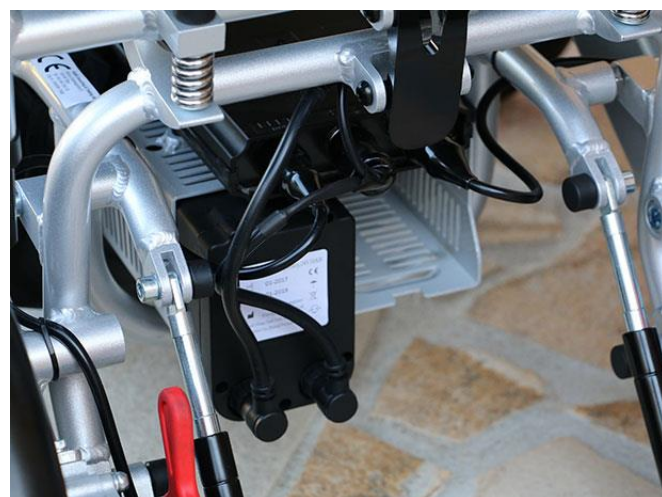

- 6** Your FreedomChair can run with only one battery without problem.